

MedGyn Postpartum Balloon Catheter

The MedGyn Postpartum Balloon Catheter is a disposable, multiple lumen catheter attached to an inflatable balloon system designed to provide tamponade for controlling hemorrhage from the uterus and vagina.

Features & Benefits

1. 100% silicone material allows for patient comfort.
2. Pear-shaped balloon design is perfectly fit for uterine anatomy.
3. Dual lumen catheter allows inflating balloon and monitoring hemorrhage.
4. Two syringes for inflating balloon rapidly.
5. Bag spike is equipped with check valve.
6. Three-way stopcock for easy, safe and rapid inflation.
7. Balloon can intake up to 5000 ml saline water (never with gas).
8. Includes 24 fr balloon tamponade, 59 cm catheter with three-way stopcock, bag spike, and two 50 mL luer-lock syringes.




Syringes



Three-Way Stopcock



Bag Spike

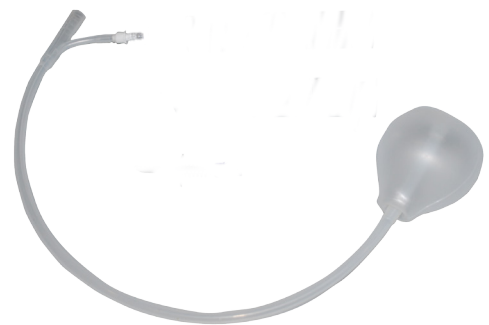

Dual Lumen Catheter and
Pear-Shaped Balloon




Balloon Inflation and Placement



Option 1: Direct Saline Inflation

Spike a 500 ml bag of fluid. Prime the tubing by attaching the syringe and drawing fluid. Using three-way stopcock connect syringe to balloon catheter. Push fluid into the balloon.

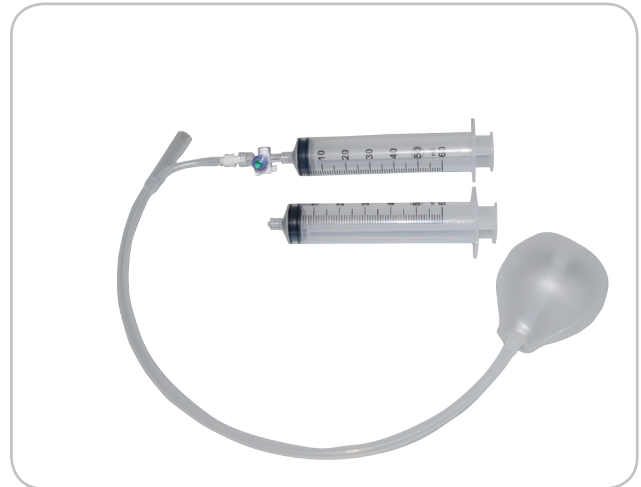

Option 2: Indirect Saline Inflation

Draw saline fluid into syringe. Connect syringe to balloon catheter. Push fluid into the balloon. Two 60ml syringes available to inflate balloon.

022703 MedGyn Postpartum Balloon Catheter

TRUSTED PRODUCTS FOR PREMIER WOMEN'S HEALTHCARE

MEDGYN PRODUCTS, INC.

100 W. Industrial Rd. Addison, IL 60101 USA

t: +1 630.627.4105 toll-free: 800.451.9667

f: +1 630.627.0127

e: info@medgyn.com

w: medgyn.com

